

SUCTION CLEANER

SURGICAL EVACUATION SYSTEM CLEANER

YOU CAN CLEAN WITHOUT DISINFECTING, BUT YOU CANNOT DISINFECT WITHOUT CLEANING.

SUCTION CLEANER containing Quadex® (V6) is a HIGH-LEVEL CLEANSING CONCENTRATE that rapidly cleans and deodorises the entire evacuation system. Used daily, this superior evacuation system cleaner will keep lines running free and clear at maximum suction; protects evacuation systems and helps to extend its life. Clinically proven to clean lines and discourage build-up and to break down and dissolve bacteria-laden solid debris like coagulated blood, saliva, prophy paste, or impression materials. Eliminates odours and has a fresh lavender fragrance and is safe and gentle on all plastic piping at correct dilution levels. Used for all suction and vacuum pumps utilised for nasal suction, oral suction, nasopharyngeal and oropharyngeal suction, or deep suctioning, as well as surgical evacuation systems.

DIRECTIONS FOR USE:

Dilute 20ml disinfecting SUCTION CLEANER into 2 litres of warm water, then draw 250ml of the solution through each evacuation system line. It is preferable to allow the mixture to remain in the system for at least 10 minutes to optimise the disinfecting properties of the product and to dissolve semi-solid matter. REPEAT AT THE END OF EACH DAY.

Quadex® (V6) exhibits the latest features desired of a modern disinfectant.

- NON-TOXIC
- NON-STAINING
- NON-IRRITANT
- NON-CORROSIVE
- pH NEUTRAL
- FAST-ACTING
- NON-HAZARDOUS
- BROAD-SPECTRUM
- EFFICACY
- WATER BASED
- REMOVES BIOFILM
- ENVIRONMENTALLY SOUND
- NEUTRALISES (organic and inorganic odours and malodours)
- RESIDUAL (30-day laboratory tested residual - unlike most disinfectants that merely kill and evaporate)

Suction Cleaner is available in:

- 1 Litre - Easy Dose Bottle
- 5 Litre - Bulk

Fragrance:

- Lavender

Feature:

- Penetrates Biofilm

Ambit of use:

- B/S C/S D/S

GMDN Code:

- 47631



"ABSOLUTE PEACE OF MIND"

Cell: 086 111 2661 • sales@quadex.co.za • www.quadex.co.za

